# SKIN TEC

■ ■ ■ ■ ■ ■ ■ ■ BY JPR

**Triple  
Technology**  
for perfect skin results

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## SKIN TEC BY JPR ■■■

Jean-Pierre Rosselet Professional Beauty represents high-quality, effective skin care and special applications in specialist institutes. High-performance products go hand in hand here with latest generation high-tech applications. Just this combination as it stands promises the best and immediately visible results. It has given rise to a new skin care and treatment concept: SKIN TEC by JPR.

The SKIN TEC treatment system from JPR Professional Beauty unifies three technologies to provide comprehensive and effective treatment in the field of skin renewal, regeneration and anti-ageing, as well as for problem skin.

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## 1. MICRODERMABRASION ■■■

SKIN TEC by JPR microdermabrasion involves removing dead horny cells (callus) through gentle and absolutely skin-friendly mechanical movements with specially shaped sterile abrasion tips. The spring integrated in the applicator prevents the skin from being injured if too much pressure is applied. This ensures extremely high treatment safety. The abrasion effect smooths the skin, evens out the complexion and refines the pores. The absorption capacity of the epidermis with regards to subsequently applied products is significantly improved. The massaging movements of the abrasive tips also promote micro-circulation in the skin.

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## 2. MESOSTIMULATION ■■■

SKIN TEC by JPR mesostimulation is an absolutely painless and comfortable treatment technology. The uppermost layer of the epidermis is stimulated with specifically arranged conical plastic tips. The resulting micro-channels reduce the diffusion barrier of the skin surface so that specific active ingredients with the appropriate molecular proportions can be infused into the epidermis with better results. The surface micro-channels are closed very rapidly by the skin. Specific active ingredients matching skin requirements can be worked into the epidermis with this mesostimulation technology.

This technology reactivates skin functions and stimulates the repair system of the skin itself. Collagen and elastin production is enhanced, skin elasticity is improved, expression lines are reduced and facial contours appear more defined.

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### 3. ULTRASOUND ■■■

Ultrasound generates high-frequency vibrations that activate cell metabolism and support the regeneration of damaged skin. This stimulates the activity of the fibroblasts. The vibration produced by the sound expands the intracellular spaces of the skin and promotes the natural purification process. Specific active ingredients can penetrate more easily.

Various treatment techniques are made possible with the ultrasound applicator from SKIN TEC by JPR. The deep cleansing process cleans the skin pores and impurities such as blackheads and dead skin scales are gently removed. This application technology refines the pores and the entire appearance of the skin.

During ionisation, the absorption capacity of the skin is stimulated and specific active ingredients can be infused into the epidermis. The skin is regenerated and provided with nutrients. It looks fresh and youthful again.

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The various technologies that SKIN TEC by JPR offers can be used in diverse combinations according to the needs and condition of the skin.

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